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**Thank you for purchasing the FontaFit Tavi. Please read the operating instruction carefully and follow the instructions and tips to get the most out of your Fitness Bracelet.**

## Package content

Smart Bracelet | Charging cradle | Manual

## Overview

1. Bracelet with quick release
2. Housing
3. 1.22" TFT Display
4. Touch sensitive button
5. Sensors
6. Charging connection

## Getting started

### Charging the battery

The FontaFit Tavi is equipped with a rechargeable Lithium-Polymer battery. Fully charge the FontaFit Tavi prior to putting the device into operation for the first time. For this purpose, please use attached charger. Place the Tavi in the charger cradle where the pins of the charger cradle and the Tavi are in contact (Fig.1). Close the charger cradle. The charging cradle can be charged with any USB-A power source or the USB-A port of a Computer. Connect the USB cable to a USB power source; the battery symbol will show up and the battery will be charged. The usual charging time is 1-2 hours. As soon as the battery is fully charged, the icon will show a green Battery.

**Note:** The bracelet can be exchanged with common 20 mm bracelets with spring bar.

### Switching the device on and off

**To switch it on,** press and hold the button (4) for about 10 seconds. The FontaFit Tavi will switch on. **To power it off,** tap the button (4) until the time is displayed; then press and hold the button (4) twice for about 3 seconds. The info menu appears followed by the power off/reset menu. Tap on the button (4) to select the menu entry „Off“ and press and hold the button (4) again for about 3 seconds to confirm your selection and power off the FontaFit Tavi.

### Display

The FontaFit Tavi has a built-in LCD color display. The Date, Time, connection to the phone, walked steps, sleep duration, heart rate, blood pressure, blood oxygen, weather, push messages are shown on the display.

**Attention:** Date and time will only be updated after connecting to the smartphone. Manual configuration is not possible. Notification with emojis are not supported.

### Watch face individualization


You can customize the watch face 1 with a photo of your choice.

1. Select the menu item „Watch faces“ in the app and go to „Edit“.
2. Go to „Select picture“, select your picture and define the section.
3. Adjust the colour, position and other settings as required.
4. Click on Save and wait until the image has been uploaded to the tavi. This process takes about 90 seconds.

Three different watch faces are available. These can be selected via the app.

### Enter submenu

Tap the button (4) to display the main menu items.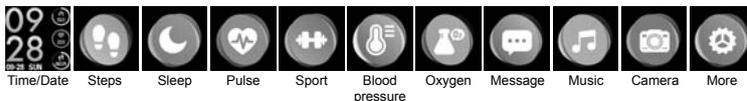

Press and hold the button (4) for approx. 3 seconds to activate the submenu (if available). Press and hold the button (4) for approx. 3 seconds in the submenu to confirm a selection or return to the main menu.

### Step count

The step count burned calories and distance are displayed.

### Sleep duration

The determined sleep duration is displayed here if you wear the FontaFit Tavi while sleeping. Further data will be displayed in the app.

### Pulse measurement

The pulse measurement is started here by press and holding the key (4).

### Blood pressure measurement

The blood pressure measurement is started here by pressing and holding button (4).

### Measurement of oxygen saturation

The measurement of oxygen saturation is started here by pressing and holding key (4).


**Please note:** Wear the bracelet approx. 1cm above your wrist bone and make sure it is close to your skin. Please note that other factors can lead to measurement inaccuracies: Low blood circulation • Tattoos • Uneven or jerky movements • Sunscreen and other lotions • dirty sensor.

**Note:** By using optoelectronic pulse measurement (photoplethysmography), in which your blood flow is detected by green LEDs and photo diodes, FontaFit also displays values when the bracelet is worn too loosely or not worn at all. For this reason, make sure that the wristband fits correctly when measuring your heart rate.

**This is not a medical device: Data from the Step, pulse, blood pressure and oxygen saturation functions are not intended to be used for medical purposes.**

## Sport

Some activities are pre-programmed in the Sports menu. To enter the submenu, press and the hold button (4). Here you can choose between Walking, Running, Cycling, Skipping, Badminton, Basketball, Soccer, Swimming. Activate them by pressing and holding the button (4). Dynamic heart rate measurement is activated in sport mode. You can end current activities by pressing and holding the button (4). Confirm your selection.

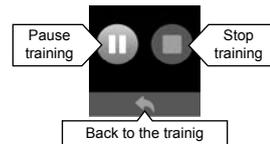## Notifications and incoming phone calls

The FontaFit Tavi displays notifications or incoming calls directly. In the menu, you can display the last 3 notifications by pressing and holding the button (4).

## Other menu items

If your FontaFit Tavi is connected to your smartphone and within range, the following menu items are available:

- Current weather and 1-day weather forecast
- camera remote control
- control music.

This can be activated in the main menu by pressing and holding the button (4).

## Installing the app

Download and install the „FontaFit Pro“ app by scanning the QR code or follow the link:(URL: [www.d-parts.de/app/fontafit\\_pro](http://www.d-parts.de/app/fontafit_pro))

**Note:** The Android app requires Android 4.4 or later. The iOS app requires iOS 9 or later.

## Establishing a Bluetooth® connection and using the app

Switch on FontaFit Tavi, enable Bluetooth® on your smartphone and start the app. First enter your data in the app, such as your gender, height, weight and year of birth.

**Note:** This data is used to correctly calculate calorie consumption and distance.

Go to „ADD DEVICE“ to connect the FontaFit Tavi to your smartphone. After successful connection, date, time and weather data are transferred from the smartphone to the FontaFit Tavi.

## The following functions can be activated by the app:

- Watch face: Here you can choose between three watch faces which are displayed on the FontaFit Tavi.
- Notifications: Here you set which notifications are displayed on the FontaFit Tavi.
- Alarm clock: Here you can set up to 3 alarm clocks. The FontaFit Tavi will vibrate when the alarm clock rings.
- Photo Shutter: Here you can use the FontaFit Tavi as a photo shutter for the camera of your smartphone.
- Reminder: If you activate this function, the FontaFit Tavi will remind you every hour by vibrating. You can use the reminder function to get up, walk, drink, etc.
- Do not disturb: here you can set the time at which no notifications are sent to the FontaFit Tavi.
- Full-time Heart Rate: When enabled, Tavi automatically measures your heart rate every 30 minutes.“

For more information on steps, sleep, pulse, blood pressure, oxygen, sport activities and route recording, tap the appropriate menu in the app.

**Note:** The route recording requires activation of the GPS function of the smartphone.

If you need technical support, please contact our support team by our website under [support.fontastic.eu](http://support.fontastic.eu).

## Safety advice:

- Do not open the device
- The device is water-resistant, not waterproof. Further information on water resistance can be found on our website [www.d-parts.de](http://www.d-parts.de).
- Do not use the device when it has visible defects.

## Technical Data

- Connection: Bluetooth® 4.0
- Battery type: Lithium polymer
- Display: 1.22" TFT color Display
- Working temperature: -10° - +40° C
- Material Bracelet: Silicone
- IP68 protection for continuous immersion up to 30 minutes in 1.5 meter depth

## Registration and Safety

**CE** Certification/General Information  
We hereby declare, that this device carries the CE mark in accordance with the regulations and standards. It conforms with the fundamental requirements of the (RED) 2014/53/EC.

## Note on environmental protection:

After the implementation of the European Directive 2012/19/EU in the national legal system, the following applies:

Electrical and electronic devices may not be disposed of with domestic waste. Consumers are obliged by law to return electrical and electronic devices at the end of their service lives to the public collecting points set up for this purpose or point of sale. Details of this are defined by the national law of the respective country. This symbol on the product, the instruction manual or the package indicates that a product is subject to these regulations. By recycling, re-using the materials or other forms of utilising old devices, you are making an important contribution to protecting our environment.